


# EPI-BLEND Plus

## Permanent hair removal device

### Instructions for use

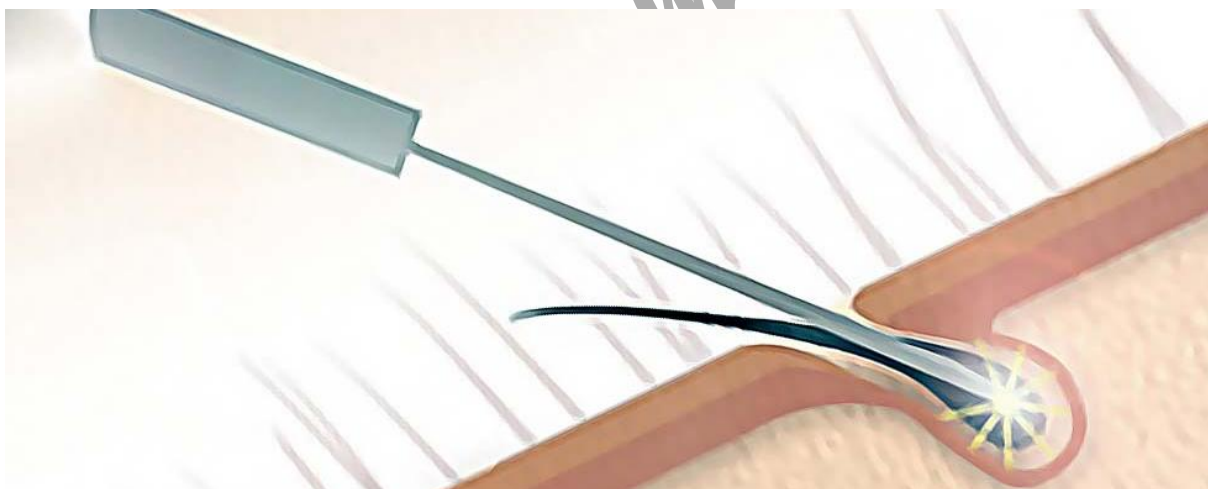

Manufactured and distributed by:

***ELKONcosmetic Kft.***

**H-2143 Kistarcsa, Szabadság út 54.**

**Phone: +36-28-472129**

**[www.elkon.hu](http://www.elkon.hu)**

**[elkon@elkon.hu](mailto:elkon@elkon.hu)**

## Description of controls:



## Accessories:



## Dear customer!

The **EPI-BLEND Plus** a microprocessor permanent hair removal device that includes all the beneficial features of its predecessors, many years of manufacturing experience, and the recommendations of beauticians experienced in hair removal.

### Main features:

- Easy handling
- Maximum efficiency
- Graphic LCD display
- Patient protection
- Electronic protections to avoid malfunctions
- Long life

The philosophy of ELKONcosmetic Kft. is to use uncompromising modern circuit elements and solutions in its developments and devices.

The **EPI-BLEND Plus** the "soul" of the device is a microprocessor, which tirelessly monitors the individual parameters and the patient so that the most optimal and effective treatment can always be achieved with it.

All information can be found on the graphic LCD display, which greatly facilitates the effective use of the device.

### Purpose of the device:

The device is designed for professionals (cosmetologists) and can be used for professional cosmetic treatment. The device combines the ELECTROLYSIS and THERMOLYSIS methods, the greatness of which lies in the fact that a galvanic current (BLEND) and a high-frequency heat effect (THERMO) are created at the same time. So the positive properties of both methods are combined, so the result is permanent hair removal in a short time!

### About epilation in general:

Hair growth has three phases.

1. ANAGEN, i.e. growth phase. This is where the hair follicle is located the deepest.
2. CATAGEN, i.e. declining phase. The root of the hair gradually atrophies. The hair follicle is located less and less deeply.
3. TELOGENE, i.e. resting phase. Most hairs that have never been treated or have not been treated for a long time. The hair follicle is almost on the surface of the skin.

In the ANAGÉN phase, the needle can easily be inserted to the depth of the hair follicle. We can effectively treat the fiber in the ANAGÉN (growing) phase, when it is still tightly attached to the connective tissue, has a supply of nutrients, i.e. is alive.

The needle must be inserted parallel to the hair fiber, up to the hair follicle, in which case the operation is painless and does not involve bleeding.



**Course of treatment:**

Connect the accessories to the device. Turn on the device with the switch on the back.

The welcome text appears on the display. Press the OK button, the adjustable parameters can be seen on the display.

Insert the rubber electrode of the inactive conductor into the moistened pad.

Attach the inactive electrode prepared in this way to the right arm of your guest with the fastening strap.

Insert the epilation needle into the needle holder of the active electrode. Use the navigation buttons to set the desired treatment values.

By pressing the OK button, the treatment time starts counting down.


Insert the needle next to the hair to be removed.

Grasp the hair to be removed with tweezers.

Use the foot switch to start the pulse, then and at the end of the pulse the device will make a "squealing" sound. At the end of the pulse, pull out the hair with tweezers.

The pulse counter shows one more with each pulse start, so you can control the removed hairs and thus calculate the treatment fee.

At the end of treatment, you can reset the counter by pressing the OK button.

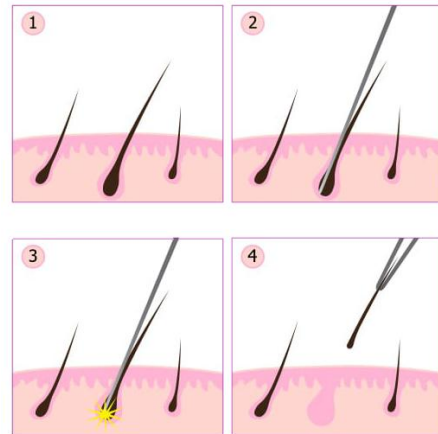**Package contents:**

Appliance

Active wire

Electrode with needle clamp

Inactive wire

Intermediate pillow

Attachment strap

Foot switch

**Specifications:**

Operating frequency 0.5MHz

BLEND current 0-2.5mA

Pulse time 0.1-10 sec Mains

voltage 230VAC

Consumption 20VA

Manufactured and distributed by:

***ELKONcosmetic Kft.***

**H-2143 Kistarcsa, Szabadság út 54.**

**Phone: +36-28-472129**

**[www.elkon.hu](http://www.elkon.hu)**

**[elkon@elkon.hu](mailto:elkon@elkon.hu)**